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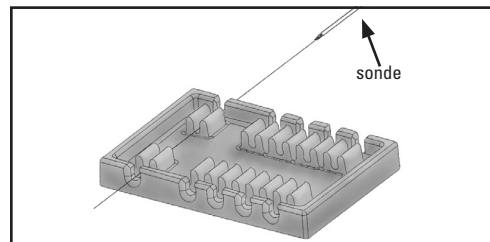
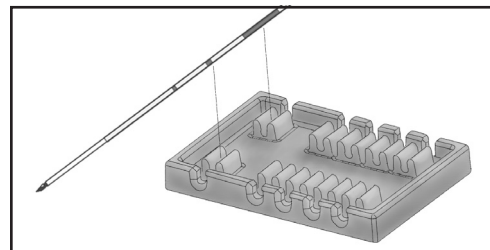
CE 2797

16910474-01B / 2023-03

© 2023 AngioDynamics, Inc. or its affiliates. All rights reserved.

UK Responsible Person:
AngioDynamics UK Ltd.
29 Harbour
Kidd Rapinnet
Exchange Square
London E14 9GE
United Kingdom

AngioDynamics
Netherlands BV
Haaksbeekweg 75
1101 BR, Amsterdam
The Netherlands

AngioDynamics, Inc.
603 Queensbury Avenue
Queensbury, NY 12804 USA
USA Customer Service
800-772-6446

EN INSTRUCTIONS FOR USE

CAUTION

Rx ONLY Federal Law (USA) restricts this device to sale by or use under the order of a physician.

This device is sterilized by ethylene oxide. The device is sterile and intended for single patient use. Sterile unless the package is opened or damaged. Do not re-sterilize.

The Spacer is intended for use with AngioDynamics Single Electrode Probes only. Refer to the Probe Instructions for use for additional information regarding the NanoKnife procedure.

WARNINGS

Reuse of single-use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness, or death of the patient.

Reprocessing may compromise the integrity of the device and/or lead to device failure.

PRODUCT DESCRIPTION

The spacer is a single component with one primary probe location and 4 secondary probe locations allowing the user to position probes from 10 mm to 25 mm increments, +/- 1mm.

INTENDED USE

The Single Electrode Probe Spacer is intended to hold Single Electrode Probes parallel and at a set distance apart.

INSTRUCTIONS FOR USE

1. Inspect all devices and packaging for damage prior to use. Do NOT use any devices that are damaged or if the sterile barrier is breached.
2. Using sterile technique, open the package and carefully remove the device.
3. Attach the Spacer to the first probe by either sliding the probe through (as shown in Figure 1) or snapping it onto the probe shaft (as shown in Figure 2).
4. Attach the second probe to the Spacer by either sliding the probe through (as shown in Figure 1) or snapping it into the probe shaft (as shown in Figure 2).

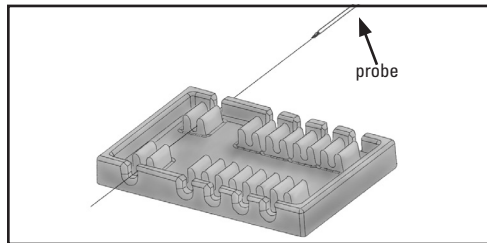
Figure 1

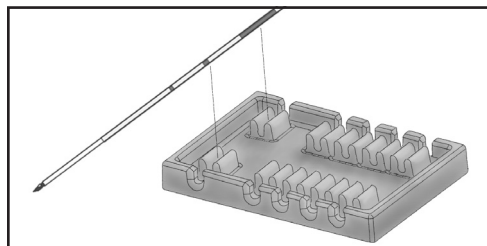
Figure 2

In compliance with the requirements of 21 CFR Part 801.15, a glossary of symbols which appear without accompanying text within the labeling of the NanoKnife Probe Spacer is provided in the NanoKnife System User Manual.

SYMBOL	REF NUM	TITLE OF SYMBOL	MEANING OF SYMBOL
UPN	NA	Universal Product Number	A Universal Product Number (UPN) code represents the manufacturer's number for an item.
	NA	Quantity in package	To indicate that the adjacent number reflects the number of units contained in the package.
	NA	Recyclable Package	Recyclable Package.*

a. EN ISO 14021 Environmental labels and declarations. Self-declared environmental claims (Type II environmental labeling)

*AngioDynamics, the AngioDynamics logo, NanoKnife and the NanoKnife logo are trademarks and/or registered trademarks of AngioDynamics, Inc., an affiliate or a subsidiary.

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